What Other Patients Have Said About MOD®


"I like to know that I do have control of my pain medication and can keep ahead of the pain. Once the pain has started and settled in, it is harder for the pain medication to be effective. At least it is so in my case."

"I love that thing, well worth its weight in gold."

"I think it's an excellent device and many people will benefit."



If you have a comment about your experience with MOD®, let us know! Visit www.avancen.com.

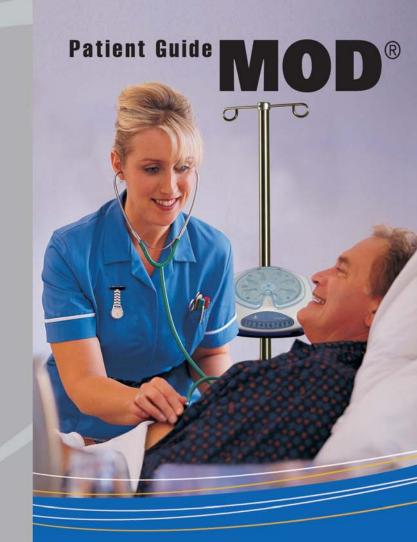

for better pain management peace of mind



What is MOD®?

Managing your pain is critical to your recovery. That's why your doctor has ordered the MOD® device for you. MOD® means Medication on Demand. There's no more waiting for your medication to be delivered ... it's already at your bedside!

MOD® is programmed so you can get your pain pill at the right time, every time.

If your doctor has written an order for you to receive a pain pill every three hours, the MOD® device will make a pain pill available to you every three hours. If you decide you're not quite ready for your pill, MOD® will be ready when you are ... but not before the prescribed time.

MOD® tracks your pain levels so your doctors and nurses can better monitor and help control your pain.

You must always record your pain level before you can access your medication. If you don't register your level of pain, the MOD® won't dispense your medication!



Prior to obtaining your pain medication, you must first touch the number that best represents your pain level.

How do I use MOD®?

The device is very simple to use. When the small light on the MOD® glows green, you have access to your pain pill.



1. Begin by touching a number on the front of the MOD® that best describes your pain level.



2. Slowly place your wristband at the center of the MOD®'s pain scale panel to open the medication tray.



3. When the tray opens, take out your pain pill. You have 45 seconds to remove your pill.